



Medical

Pall Kleenpak™ Water Filter

In-Line Disposable Water Filter with Quick Connect Inlet and Outlet Ports (KAQIN) – 31 Day Use

For Use

The Pall Kleenpak In-line Water Filter (KAQIN) is indicated for applications such as water intake lines of ice machines, kitchen equipment rinsing or other similar uses.

For full details, please refer to the Instructions for Use.

- Maximum operating pressure: 5 bar g (75 psi)
- Maximum influent water temperature 60 °C

Ordering Information

Reorder Code	Description	Packaging
KAQIN2	Pall Kleenpak Water Filter – In-line Disposable Water Filter with Quick Connect Inlet and Outlet Ports	2 units per case
KAQIN	Pall Kleenpak Water Filter – In-line Disposable Water Filter with Quick Connect Inlet and Outlet Ports	12 units per case

Quick Connect Adaptors


Code	Description	Packaging
TAP1/2MN	½ inch, male thread, non-valved	1 unit
TAP1/2MV	½ inch, male thread, valved	1 unit
TAP1/2FN	½ inch, female thread, non-valved	1 unit
TAP1/2FV	½ inch, female thread, valved	1 unit
TAP22FN	22 mm, female thread, non-valved	1 unit
TAP22FV	22 mm, female thread, valved	1 unit
TAP24MN	24 mm, male thread, non-valved	1 unit
TAP24MV	24 mm, male thread, valved	1 unit
TAPUC	Universal (15 – 29 mm), non-threaded	1 unit

*For information on suitable adaptors, contact your local Pall representative.



Vertrieb durch: Filteria.de
 Medizinischer Handel
 Jürgen Single
 Schulze-Delitzsch-Str. 15
 71706 Markgröningen
 Tel.: 07145-991 66 66
 Fax: 07145-991 66 69
 eMail: kontakt@filteria.de
 Internet: www.filteria.de

Quick Connection, Immediate Protection.

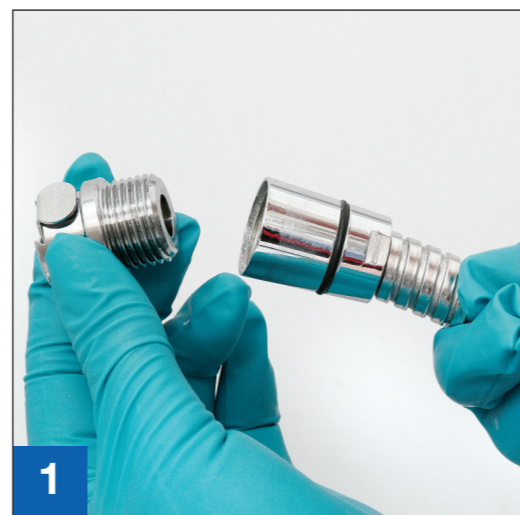

- 1
- Select appropriate Quick Connect Adaptors* and attach to the upstream and downstream water lines.
 - Attachments should be sanitised and connections made aseptically.



- 2
- Handle filter aseptically.
 - Remove protective cap from filter outlet using a twisting motion.
 - Ensure correct filter orientation prior to making any connections (refer to flow arrows on filter label).



- 3
- Ensure the black o-ring is positioned on the filter outlet prior to connecting to the downstream water line.
 - Push filter outlet into quick connect adaptor.



- 4
- Ensure the black o-ring is positioned on the filter inlet prior to connecting to the upstream water line.
 - Push filter inlet into quick connect adaptor.
 - Complete the time on (☉) time off (☿) indication as appropriate.



- 5
- Pall Kleenpak In-Line Disposable Water Filters (KAQIN) may be used for up to 31 days. To remove the filter, push the thumb latch each connector adaptor to release.



Pall International Sàrl
 Avenue de Tivoli 3
 1700 Fribourg, Switzerland

LifeSciences.EU@pall.com email

Visit us on the Web at www.pall.com/healthcarewater

This document is not for distribution in the USA and Canada.
 The information provided in this literature was reviewed for accuracy at the time of publication. Product data may be subject to change without notice. For current information consult your local Pall distributor or contact Pall directly.

© 2010, Pall Europe Ltd. Pall, PALL and Kleenpak are trademarks of Pall Corporation. ® indicates a trademark registered in the USA and TM indicates a common law trademark.

11/10, PDF, GN10.3868